

Sharpsafe Label



Once you have assembled the Sharpsafe container don't forget to fill in the details on the label.

Sharpsafe Temporary Closure

To release the temporary closure feature press down on the tab, rotate the lid until it engages in the open position the container is now ready for use.




To engage the temporary closure feature release lid from the open position rotating the lid until it covers the aperture, continue to close the lid until a click is heard. This indicates the temporary closure is engaged.

Location, Location, Location



Ensure the Sharpsafe container is placed into a stand or onto a suitable surface ready for use. The container can also be placed onto a wall mounted bracket or Sharpsafe trolley depending on your requirements.

Sharpsafe Assembly

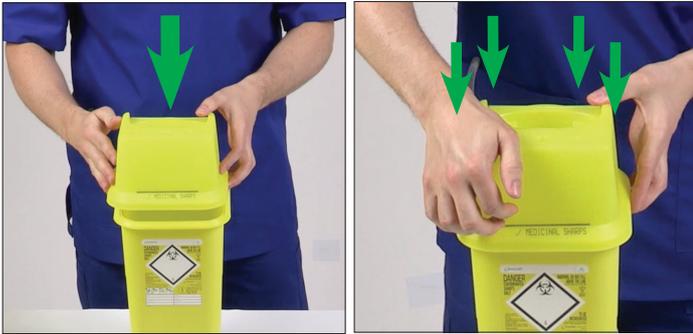

Fit the lid and base together ensuring that the line and label are on the same side.

When the lid and base are aligned apply pressure to each corner in turn until four clicks are heard. Ensure each corner is secured before use.

Sharpsafe Label



Once you have assembled the Sharpsafe container don't forget to fill in the details on the label.

Sharpsafe Temporary Closure

To release the temporary closure feature press down on the tab, rotate the lid until it engages in the open position the container is now ready for use.



To engage the temporary closure feature release lid from the open position rotating the lid until it covers the aperture, continue to close the lid until a click is heard. This indicates the temporary closure is engaged.

Sharpsafe Closure

Never overfill a Sharpsafe container.

The container should be closed and sealed in accordance with your local infection control protocols.



Place the container onto a suitable surface to engage final closure. With the container in temporary closure, press down firmly to engage final closure. Ensure all **3 clicks** are heard to indicate final closure. Fill in the details on the label with name and date the container was sealed.